Print Size: 445mm x 122mm

Fold Line

Fold Line

Fold Line

Fold Line

Fold Line

Fold Line

Fold Line

Fold Line

Fold Line

Barrier Cream
Schutzcreme
Crème de protection
Crema de Barrera
barrierecreme
Suojavoidetta
barriärsalva
Crema Barrera
Κρέμα Φραγμού

Beschermende Huidcrème
Crema Barreira
Krem Medihoney™ Barrier Cream
كريم محافظ مدى حاجتي
Barrier Krem
كريم ميهيوني الحمايل
Zaščitna krema
Medihoney™-barrierekrem
Krem odronny

Ingredients: Purified water, Manuka Honey, Coconut Oil, Natural Triglyceride (Caprylic/Capric Triglyceride), Silicone Fluid 593, Cetomacrogol 1000, Cetostearyl Alcohol, Bees Wax White, Evening Primrose Oil, Sodium Benzoate, Aloe Babadensis, D-Alpha Tocopherol Acetate, Vitamin E (D-Alpha Tocopherols), Chamomile (Matricaria Recuteta), Xanthum Gum

Product components from New Zealand
Made in Canada

<ul style="list-style-type: none"> Derma Sciences, Inc. 104 Shorting Road Toronto, Ontario M1S 3S4 USA Int: +1 609 514 4744 Canada Int: +1 416 299 4003 	<p>SC RECIPE</p> <p>Scanlan Group B.V. 1118 ZS, Schiphol Tripont The Netherlands, Int: +31 20 653-0553</p> <p>Australian Sponsor Emergo Australia 20 Sussex Street Darling Park, Tower 11, Level 20 Sydney, NSW 2000 Australia Int: +61 2 9006 1662</p>
----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

www.dermasciences.com

MEDIHONEY® is a trademark of Medihoney Pty Ltd.
Licensed by Comitta New Zealand Ltd.

R10393 REV.2

Medihoney® Barrier Cream can be applied to intact and at-risk skin to provide protection from body fluids and moisture. It can be applied around wounds and under skin folds where a protective barrier will help prevent skin breakdown. The cream can also be applied under wound dressings and incontinence pads where the skin can be at-risk of damage. Medihoney™ Barrier Cream helps maintain the skin's barrier properties and maintain skin moisture.

Indications for Use

- Protects at-risk skin from breakdown associated with incontinence and can be used under incontinence pads
- Can be used around wound edges to protect the skin from irritation or breakdown caused by wound exudate
- Helps protect the skin from damage caused by friction or shear
- Helps prevent maceration
- Helps prevent excoriation
- Helps maintain the skin pH
- Helps prevent damage to skin caused by frequent hand washing

Precautions

When using for the first time, patch test on a small area of skin. If irritation occurs discontinue use.

Instructions for use

Apply to clean, dry skin as required. Reapply after bathing or at each dressing change.

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