Print Size: 445mm x 122mm

Derma Cream

Crème dermo-protectrice

Crema Derma

styrkende hudcreme

Ihovoide

Derma-salva

Dermocrema

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Medihoney™-hudkrem

Krem do skóry

Product components from New Zealand
Packaged and sterilized in Canada
Derma Sciences, Inc.
104 Shorting Road
Toronto, Ontario M1S 3S4
USA Int: +1 609 514 4744
Canada Int: +1 416 299 4003

Derma Sciences Europe Ltd.
Suites 2b & 2c
1A Park Street
Maldenhead SL6 1SL
U.K.
Tel: +44 (0) 1628 625 916
Fax: +44 (0) 1628 671 112

EC REP

Scanlan Group B.V.
Postbus 75664,
1118 ZS, Schiphol Tripport
The Netherlands,
Int: +31 20 653-0553

Australian Sponsor
Emergo Australia
201 Sussex Street
Darling Park, Tower II, Level 20
Sydney, NSW 2000 Australia
Int: +61 2 9006 1662

www.dermasciences.com

Fold Line

Medihoney® Derma Cream helps protect vulnerable skin. It can be used on large skin areas and under compression or wound dressings to protect intact skin from breakdown and irritation caused by dryness, friction or shear. It also helps moisturise dry skin.

Indications for Use

- Protects intact, vulnerable or at-risk skin from breakdown
- Helps maintain the skin's barrier properties and maintain skin moisture
- Helps protect the skin from irritation or breakdown caused by dryness, friction or shear
- Helps maintain the skin pH
- Helps prevent folliculitis
- Helps reduce itching

Precautions

When using for the first time, patch test on a small area of skin. If irritation occurs discontinue use.

Instructions for use

Apply to clean, dry skin three times a day or as required. Reapply after bathing or at each dressing change.

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